



watar

PetPal

CDV Rapid Test Kit



Sensitive Accurate Fast Easy

SP002 10Test/Box

PRINCIPLE



This rapid test strip is based on “sandwich” method to test the CDV in canine nasal drops, saliva, conjunctival secretions, serum and urine. When add the sample into the sample hole, the CDV in the sample will combined with the specific monoclonal antibody labeled by colloidal gold, and inhibit the antibody combing with the CDV -BSA in the TEST line in the NC membrane. This leads the color intensity of the test line and control line to be different. C line shows the strip is valid, T line shows there are Canine Distemper Virus in the sample.

STEPS




1. Collect the sample




2. Dissolve the sample in the tube, 1min




3. Take the supernatant

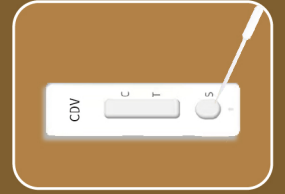


4. Add 3 drops into the orifice

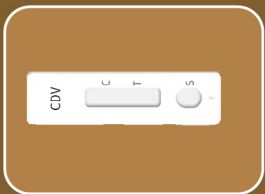

5. Incubate 5-10min, Room temp

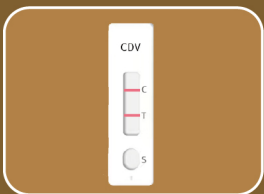

6. Read the Result

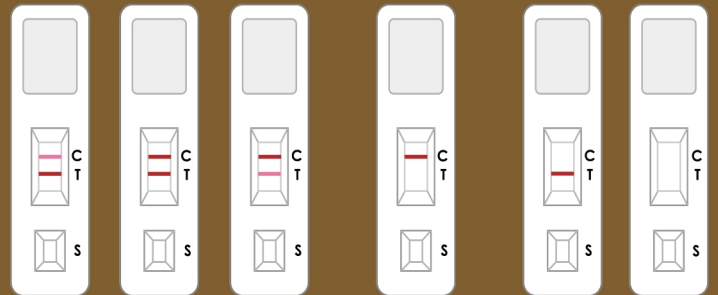

Positive

Negative

Invalid

Shaanxi Watar Biotechnology Co.,Ltd.

Add: No.12, Hongzhuan South Road, Yanta District, Xi'an 710061, China

Tel: 0086—18612182039 Service Hotline: 0086-13679259477

Web: www.watarbio.com www.watar.en.alibaba.com

Email: info@watarbio.com



Intelligence for Life Science



Store at 2-30 C, do Not freeze
Valid for 12 months

